

MW EVA 23S

DESCRIPTION:

The tactical evacuation kit is a quick, easy to assemble light weight piece of equipment that allows a combatant to rapidly transport an injured person. The device is constructed out of extremely durable carbon fiber and fitted into a carrying bag that can be attached to the combatant's load bearing equipment.

The light weight and small size of the product make it suitable for use in particularly demanding environments where carrying capacity, muscle strength and reaction time can be the difference between a life lost or a life saved.

PROPERTIES:

Length while packed: 400,00 mm

Width while packed: 130,00 mm

Length while opened: 2150,00 mm

The width of the fabric while opened: 510,00 mm

The length of the fabric while opened: 1850,00 mm

Weight without carrying bag: 2,89 kg

Maximum carry weight: 150,00 kg / 330,6 lb

Material: Carbon fiber

Functioning mechanism: Telescoping pipe structure

Fabric material: 75% PA/25% PU/~120g/m²

Carrying bag: MLLC attachments, IRR treatment which reduces the visual signature of the bag to resemble the intended environment when viewed through night vision cameras.



mediwear

sales@mediwear.fi
+358 10 201 6500

Instruction video how to fold the stretchers/YouTube: MediwearLtd
<https://www.youtube.com/watch?v=aFEpqAGIXQE>


Pat. pending FI 20235549



mediwear

MW EVA 23S MANUAL

REMOVE THE STRECHER FROM THE CARRYING BAG

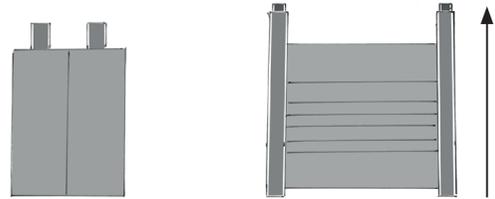

1. ROLL OPEN

2. OPEN THE TELESCOPE
STRUCTURE



4. PULL OUT HANDLE BARS



5. PULL THE HANDLE BARS ALL THE WAY OUT



3. PULL FIRMLY OPEN




6. CONNECT CARRYING STRAPS



7. CARRYING STRAPS IN USE

ALWAYS USE THE PATIENT BELT AT ALL TIMES

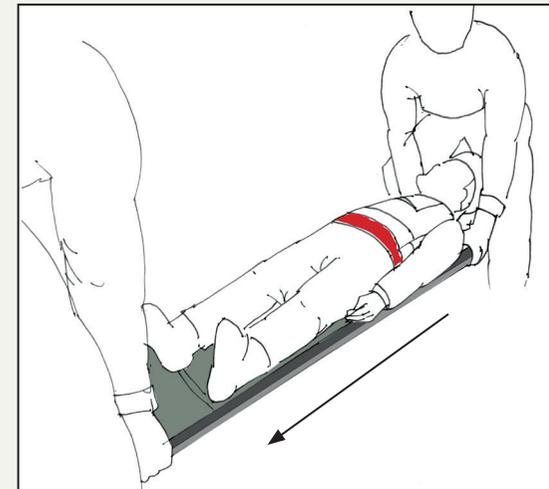

8. CARRYING DIRECTION